

**BIO** CLINICAL® **PLUX**  
**protocol quickguide**


**masseter**  
equilibrium





## Step 1


Press 'options' to setup the exercise.

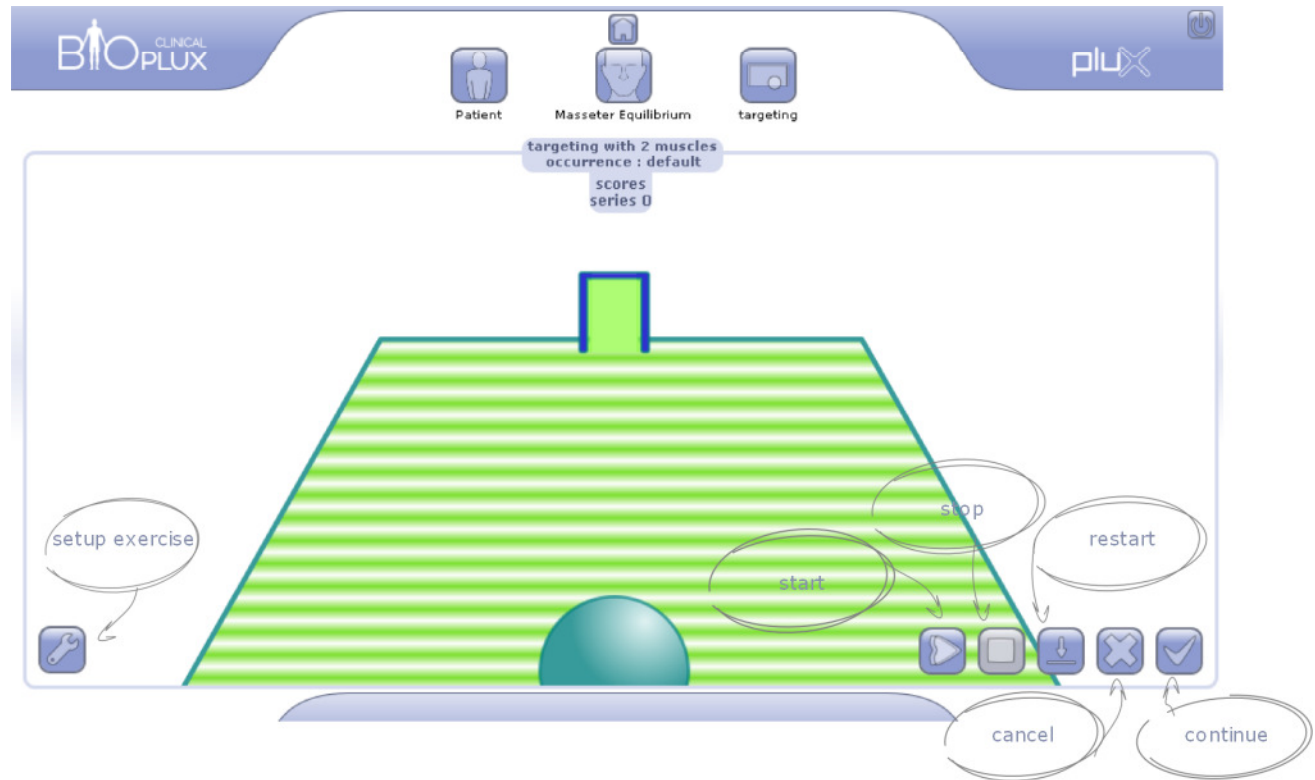



## Step 2

Here, you can define: number of repetitions; rest time (in seconds); action time (in seconds); and acceptance margin.

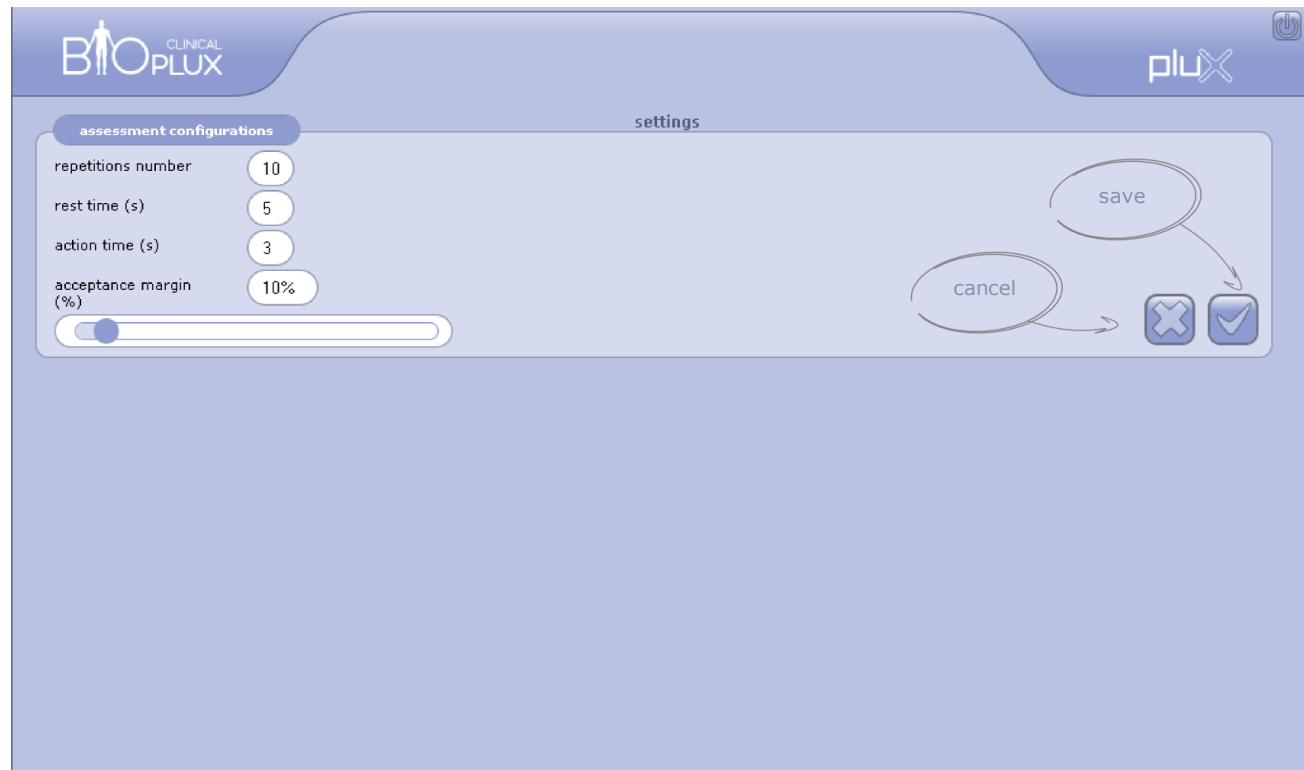



### Step 3

Press 'start'. You will see a countdown of rest time. The patient should wait until the exercise starts.





## Step 4

Start the exercise. The objective is to contract the masseter muscles in a symmetrical way, trying to score, putting the green ball in the goal net.

The goal can be changed in size, changing the difficulty.

When the resting time ends, the patient should start the exercise.

BIO PLUX CLINICAL

Patient Masseter Equilibrium targeting

pluX

targeting with 2 muscles  
occurrence : default  
scores series 1

1	2	3	4	5	6	7	8	9	10
✓	✗	✓	✓	✗	✓	✓			

goal

result of the obtained goals

ball

Wrench icon, Play icon, Stop icon, Download icon, Close icon, Checkmark icon





The screenshot displays the BIO PLUX CLINICAL software interface. At the top left is the logo 'BIO PLUX CLINICAL' and at the top right is 'plux'. The main content area is titled 'report' and contains the following elements:

- Report Summary:** A table with columns 'action time (s)' (value: 3) and 'acceptance margin (%)' (value: 30). To the right is a circular gauge showing a 'score (%)' of 80.
- Series 2 Data:** A row of 10 numbered items (1-10) with green checkmarks or red crosses below them. The sequence of results is: ✓, ✗, ✓, ✓, ✗, ✓, ✓, ✓, ✗, ✓.
- Comments Section:** A large text area labeled 'space for comments'. To its right is a field labeled 'name of the technician' with a 'technician' input field below it.
- Action Buttons:** At the bottom right, three icons are present: a trash can (labeled 'delete report'), a printer (labeled 'print report'), and a checkmark (labeled 'conclude').



[www.bioplux.com](http://www.bioplux.com)